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ZF730.01
April 2015

ISTRUZIONI D'USO

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Quadrant Total Etch

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Quadrant Total Etch

Description

Quadrant Total Etch is a 35% phosphoric gel for use with the total etch technique.

Composition

Quadrant Total Etch contains
▪ 35% phosphoric acid

Indications

▪ Conditioning of enamel and dentin according to the total etch technique.

Interactions with other materials

Base lining materials like calcium hydroxide might be dissolved by the acid, depending on their composition. Use acid proof materials only and make sure they have completely set, before applying **Quadrant Total Etch**.

Clinical procedure

1. Prepare the cavity according to the general directions for composite restorations. Clean and dry the preparation. The use of a rubber dam is recommended.
2. Apply **Quadrant Total Etch** gel onto the enamel and dentin to be restored and leave for 15-30 seconds (total etch).
3. Rinse **Quadrant Total Etch** gel thoroughly for at least 5 seconds and remove excess moisture from the preparation with a gentle stream of air. Make sure no etch is remaining at the preparation. If you wish to visually verify that enamel etching has taken place, more intense drying of the preparation is necessary. The enamel surface will show frosted-white when sufficient etched. In case of fluorized enamel, it might be necessary to repeat the steps 2 and 3.
4. Apply bonding system (i.e. **Quadrant UniBond**, **Quadrant Uni-SE-Bond** or **Quadrant Uni-1-Bond**) according to manufacturers instructions.

Safety advice

▪ **Quadrant Total Etch** contains 35% phosphoric acid. Contact with eyes and skin may cause sensitisation. In case of accidental contact with skin or eyes immediately rinse with copious amounts of water and consult a physician if necessary. To prevent sensibility contact with the skin should be avoided.

Storage advice

Do not use the material after the expiry date. Do not store at temperatures higher than 25 °C (77 °F). Always close the syringe tightly after use. Keep out of reach of children.

Packaging

Quadrant Total Etch: syringe à 2,5 ml.

Made by Cavex Holland BV
For dental use only.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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