INSTRUCTIONS FOR USE

Cavex Non Gamma-2

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The amalgam made of Cavex Non Gamma-2 is free from the corrosion-prone and weak gamma-2 phase. Durable restorations with a lasting lustre can be made.

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Indication

Stress-bearing fillings in posterior teeth (Class I and II).

Contraindications
L'usage d'amalgam n'est pas recommandé when a physician has deemed it unsuitable for the patient.

L'usage d'amalgam n'est pas recommandé for the treatment of deciduous teeth, children under 15 years old, pregnant or breastfeeding women, except when strictly deemed necessary by the practitioner on the ground of specific medical needs of the patient (from 1 July 2018).

• Hypersensitivity (allergic reaction) to amalgam components.

In such cases the use of suitable alternative materials must be considered.

Warnings
After placement or removal of dental amalgam restorations increased mercury concentration in blood and urine has been observed. According to available scientific knowledge this increase has not been associated with any adverse health effects.

• If amalgam restorations cause any adverse effects, the user should consider replacement of the dental amalgam with another material.

In individual cases, local allergic reactions (lichenoid) have been observed. Such local reactions are due to an irritant (mercury), chemical, electrochemical or allergic nature.

In case of allergy to components of dental amalgam the use of suitable alternative materials must be considered.

Unsuitable exposure (reaction allergique) to the components of amalgam may occur. In such cases the use of suitable alternative materials must be considered.

Directions for use

Cavex Non Gamma-2 is available as pre-dosed capsules. It should be noted, that the mixing and hardening time would be somewhat different, depending on the type of mixer used: a high-energy mixer gives a faster hardening. The mixed amalgam should be a plastic, coherent mass with a shiny surface.

Mixing ratio for Cavex Non Gamma-2 is: 10 parts alloy / 11 parts mercury by weight

Cavex Non Gamma-2 in all cases 5 - 7 seconds in High-energy mixer (such as Silamat)

- I-spill (alloy/mercury 400/435 mg)

- II-spill (alloy/mercury 600/653 mg)

- III-spill (alloy/mercury 800/865 mg)

Notes:

- 1. For safety reasons, it is best to replace the empty capsules after taking out the mixed amalgam, before cleaning or disposal.
- 2. The pre-dosed capsules (Type I) are valid activating: contact between alloy and mercury occurs automatically with the capsule walls during the action of the mixer.
- 3. Storage at temperatures no higher than 25°C is recommended.
- 4. Dispose of cotton wool and used capsules in accordance with national regulations.

Insertion and condensation
After mixing, the amalgam should immediately be inserted into the cavity using an amalgam carrier. The filling should be built up in layers under adequate condensation pressure using a plugger appropriate for the size of the cavity. Do not use an ultrasonic condenser. Carefully remove excess mercury.

Finishing and polishing
Approx. 8-10 minutes after mixing, the filling can be finished. Polishing should be done in a subsequent session.

"THIS ALLOY CONTAINS ZINC: THE AMALGAM MADE THEREFROM MAY SHOW EXCESSIVE EXPANSION IF MOISTURE IS INTRODUCED DURING MIXING OR COMPACTING"

Labelling
This information is in compliance with ISO24234 and also with European legislation.

MERCURY DANGER
• May impair fertility or cause harm to an unborn child.
• Lethal if inhaled.
• Causes damage to organs in case of long-term or repeated exposure.
• Highly toxic to aquatic organisms, may cause long-term adverse effects.

Recommendations for safety precautions
Dental clinical personnel, when using silver amalgam, should exercise proper procedures to avoid contact. Personnel should be educated to avoid contact with the amalgam to mercury vapour in the dental environment. Waste amalgam should be collected for disposal in a manner, which will protect both those who handle the amalgam and the environment.

1. Treat mercury and amalgam with care. Be MERCURY-AWARE.

2. Ideally a high-energy mixer should be equipped with a protective cap to cover the capsule while mixing.

3. Small drops of spilled mercury can be collected with the aid of a "mercury-collecting forceps" or a small amount of freshly mixed amalgam that will easily absorb liquid mercury. NEVER use a vacuum cleaner.

4. Placing of amalgam and fillings must be done under water-cooling and with effective vacuum suction. The use of eye protection and mouth mask is advisable.

5. The wastewater drain of the spittoon should be equipped with an amalgam separator.

6. The amalgam on the floor must be collected and handled/recycling of the amalgam waste should be taken into consideration.

6. Excess mixed amalgam should be kept in a container, designed for the purpose, or at least in a closed container filled with photographic fixer.

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