INSTRUCTIONS FOR USE

Quadrant Flow

Description
Quadrant Flow is a flowable, light-curing, radiopaque hybrid composite.

Composition
Quadrant Flow is based on Bis-GMA monomer and contains approximately 60% w/w inorganic fillers (Barium glass, SiO₂).

Shades
A2, A3, A3.5, OA2.

Indications

- minimally invasive fillings
- base under composite restorations
- cavity lining
- repairations to composite
- splints
- retention of threads and brackets
- colour masking
- fissure sealing

Clinical procedure

Use the enclosed metal cannulae for application. These are intended for single-use.

- Prepare the cavity according to the general directions for composite restorations. Clean and dry the cavity. The use of rubberdam is recommended. When restoring exposed root surfaces, a preparation may not be necessary.
- Apply Quadrant Uni-SE-Bond, Quadrant UniBond or Quadrant Uni-1-Bond (or an other universal bonding system as desired) following the instructions for use for that product. Oxygen inhibited layer should remain intact.
- Apply thin layers of **Quadrant Flow** (max. 2 mm).
- Light cure each layer in accordance with the following table, using a suitable polymerisation lamp (e.g. Translux). An oxygen-inhibited layer is formed during the polymerisation process, which should not be touched or removed. This layer provides adhesion to the following layer of composite.

Shade	Time
A2, A3, A3.5	20 s
OA2	40 s

- Quadrant Flow** can be finished immediately after polymerisation and polished to a high gloss with fine diamond drills, flexible discs and polishing brushes.

Other instructions

- The polymerisation times indicated in the instructions are minimum times and, to obtain an optimal polymerisation, should under no circumstances be reduced.
- The cannulae supplied with the syringes are intended for single-use and should be disposed after use. Relieve the pressure on the syringe after application and replace the cap after every use.
- Do not use in combination with eugenol-based products, as these can inhibit curing.
- Quadrant Flow** can cause irritation to the eyes and skin, so avoid contact with eyes and skin. In the event of contact with the eyes, rinse immediately with plenty of water and consult a doctor.
- The product may not be used after the expiry date.
- Store at a temperature below 25°C. Keep away from direct sunlight.

Packaging units

Syringes of 1.8 g.

Made by Cavex Holland BV.
For dental use only.

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Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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CAVEX

CAVEYOUR IMPRESSION IS OUR CONCERN