



CAVEX

## Cavex ImpreSafe

1. Squeeze bottle and fill measuring cup up to 30ml
2. Pour water (always first) in reservoir till indicated stripe (app. 1 litre)
3. Pour disinfectant into reservoir
4. Place impression in basket
5. Lower the basket with the impression into the fluid
6. Leave impression for 3 minutes in the fluid for complete disinfection
7. Take impression out of fluid and rinse thoroughly with water
8. Shake of excess water and put impression in a plastic bag. Seal the bag airtight and mark the bag as "disinfected"

■ **Cavex Holland BV**  
 Fustweg 5, 2031 CJ Haarlem  
 The Netherlands.  
 Tel +31 23 530 77 00  
 dental@cavex.nl  
 www.cavex.nl

CAVEX YOUR IMPRESSION IS OUR CONCERN

YOUR IMPRESSION IS OUR CONCERN

Cavex ImpreSafe



CAVEX

## Cavex ImpreSafe

### DANGER

H302

H314

H410

P280.2

P301+P312

P301+P330+P331

P303+P361+P353

P305+P351+P338

P501.1

Harmful if swallowed.

Causes severe skin burns and eye damage.

Very toxic to aquatic life with long lasting effects.

Wear protective gloves and eye protection.

IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell.

IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

IF ON SKIN (or hair): Remove/ Take off immediately all contaminated clothing. Rinse skin with water/shower.

IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

Dispose of contents/container in accordance with relevant regulations.

YOUR IMPRESSION IS OUR CONCERN

Cavex ImpreSafe



CAVEX YOUR IMPRESSION IS OUR CONCERN

## YOUR IMPRESSION IS OUR CONCERN

Being an impression specialist, Cavex has a complete assortment of impression materials and accessories in the Cavex System. The Cavex System enables any user to work efficient, safe, with a minimum of effort and always of superior quality. **COMPLETE YOUR CAVEX SYSTEM:**



CAVEX YOUR IMPRESSION IS OUR CONCERN

# CAVEX

## Meet our silicones

### Cavex SiliconA

#### A-Silicones

- Thixotropic
- Hydrophilic
- Perfect consistency
- For all indications



### Cavex StabiSil

#### C-Silicones

- 7 days dimensional stability
- Thixotropic
- Setting time can be regulated



More information on [www.cavex.nl](http://www.cavex.nl)

HA0501.05