## CAVEX EXSENSE TOOTH CONDITIONER

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## INSTRUCTIONS FOR USE

### Instructions for use

**Cavex ExSense reduces and prevents tooth sensitivity. Its effectiveness is clinically proven.**

**DESCRIPTION**
Cavex ExSense is a water-based gel with active ingredients against tooth sensitivity; hydroxyapatite and potassium nitrate. The mint aroma adds a pleasant taste. Cavex ExSense remineralises the teeth, which makes the teeth stronger, significantly reduces sensitivity and the whiteness of the tooth is enhanced.

**VERPAKKING**
Tube 42ml / 50 grams (sufficient for around 20 treatments).

**COMPOSITION**
Key-ingredients: Hydroxyapatite 2%, Potassium nitrate 0.1%

#### INDICATIONS FOR USE

- Prevention and reduction of tooth sensitivity.
- After teeth whitening in dental practice and / or at home.
- After professional cleansing of the teeth.

#### CONTRA-INDICATIONS

Do not use if allergic to mint or one of the other ingredients.

#### HANDLING PROCEDURE

Clean teeth beforehand by brushing and flossing.

#### APPLICATION

**WITHOUT TRAY:**
Wash hands thoroughly with water and soap. Dispense a small amount of Cavex ExSense on a clean finger and apply it on the upper and lower teeth. Leave the gel on for 10 minutes. Spit out any excess gel but do not rinse afterwards.

#### WITH TRAY:

Dispense a small amount of Cavex ExSense across the entire tray. Place the tray in the mouth and move the tray a little from left to right to spread the gel on the teeth. Gently bite down on the tray and leave the tray with the gel in place for 10 minutes. Remove the tray and spit out any excess gel but do not rinse afterwards. Rinse tray with cold water.

- Always close the tube with the cap immediately after use.
- For best results: do not drink or eat for at least 30 minutes after application.
- Use Cavex ExSense once a day for a minimum of 10 days or follow the instructions of your dental professional.

#### WARNINGS & FIRST AID INSTRUCTIONS

In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention. Do not use the product after expiry date.

#### STORAGE ADVICE

Store in a cool and dry place (frost-free) and protected from direct sunlight.

### Instructions for use

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release us from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and therefore entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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