-
PREMIUM WHITENING

CAVEX BITE & WHITE PATIENT KIT

-
Whitening system

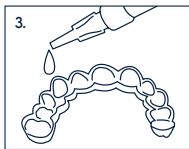
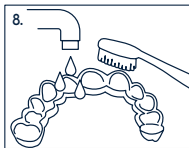
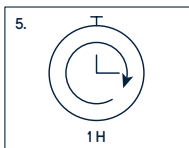
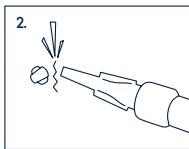
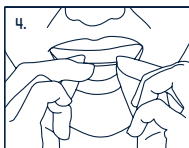
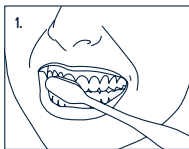
AT-HOME

INSTRUCTIONS

CAVEX

CAVEX BITE&WHITE PATIENT KIT

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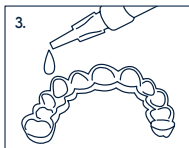
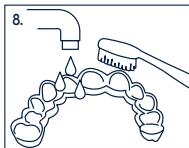
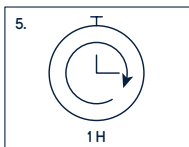
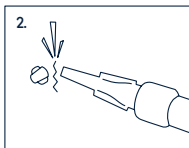
CAVEX BITE&WHITE PATIENT KIT

Instructions for use

1. Brush your teeth thoroughly
2. Break the cap off the nozzle
3. Administer 1 drop per tooth into the tray

4. Insert the bleaching tray into your mouth
5. Bleach for at least one hour
6. Rinse your mouth with water

7. Wait at least 30 minutes and then brush your teeth carefully
8. Clean the bleaching tray with your toothbrush

Hydrogen peroxide is the bleaching agent in Cavex Bite&White. It is added as carbamide peroxide in order to obtain a sufficiently stable product. The carbamide peroxide breaks down rapidly into oxygen radicals and water. During the whitening process the free radicals attack the double carbon bonds that are the source of most colours (stains) in the tooth.

PACKAGING

- Cavex Bite&White 3 ml syringes

COMPOSITION

Active ingredients: 16 % Carbamide Peroxide – equivalent to 6 % hydrogen peroxide, 0.2 % Potassium nitrate desensitizer, 0.1 % Sodium fluoride desensitizer

INDICATIONS FOR USE

Cavex Bite&White is a whitening product used to lighten the colour of natural teeth. Cavex Bite&White should only be dispensed by or on the order of a dental professional to people over the age of 18. A consultation with a qualified dentist is recommended before beginning the whitening procedure to check for any underlying oral or health issues

CONTRAINDICATIONS

Any patient falling into one of the categories below is advised to consult a doctor prior to starting the treatment:

- Pregnant and lactating women
- Those currently treated for a serious illness

CLINICAL PROCEDURE

- Record starting shade, review current dental health and make sure patient has realistic expectations about final results.
- Provisional or temporary restorations may discolour during the whitening process while composite or polycarbonate materials will remain unaffected. Therefore, patients with composite restorations must be advised that existing composites may require replacement, since only natural teeth will whiten.
- For patients requiring aesthetic or adhesive restorations, wait 2 weeks after the last whitening treatment to allow for colour stabilization. All restorations should be intact or well sealed with adequate temporary materials.
- The use of a sensitivity relief product starting 2 weeks prior to the treatment procedure until 2 weeks after the treatment should be advised.
- Patients should always be warned of tooth sensitivity or gingival irritation during the treatment. The gingival areas

CAVEX BITE&WHITE PATIENT KIT

Instructions for use

- that come in contact could turn red or white but will return to its normal colour after several days.
- To avoid soft tissue irritation, advise the patient to avoid overfilling trays, to wipe away any excess gel from the gum line and to rinse the mouth thoroughly after the treatment.
 - After each treatment, make sure patient replaces the syringe cap.
 - The patient should not eat, drink or smoke tobacco for at least 60 minutes after each treatment.
 - Instruct the patient to stop the treatment and contact you immediately if any problem develops.
 - The final shade will be achieved about 2 weeks after the start of the treatment.

WARNINGS

- Contains Fluoride.
- Contains Hydrogen Peroxide.
- Avoid contact with eyes.
- Keep out of reach of children.
- Avoid exposure of known hypersensitive teeth.
- Do not eat, drink or smoke while wearing the filled custom-fitted trays.
- Harmful when swallowed.

FIRST AID INSTRUCTIONS

- If a large quantity (over 25% of the gel in the syringe) is swallowed, immediately drink water, do not induce vomiting and contact a doctor.

- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to: **Sensitivity** Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

Gingival Irritation Trays that extend onto the gingiva may cause mild to severe soft tissue irritation. If the trays fit properly, check if the patient applies the proper quantity in the proper way. The patient may need to reduce the amount of gel in their trays.

STORAGE ADVICE

- Store refrigerated (<8°C) in a dark, dry place.
- Do not use the material after the expiry date.

DISPOSAL

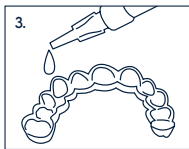
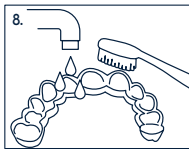
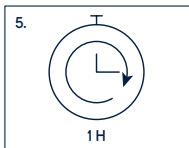
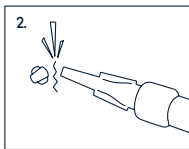
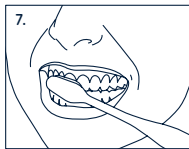
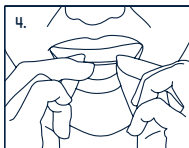
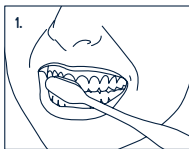
- Dispose of device in accordance with local waste regulations.

Made in The Netherlands

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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