-
PREMIUM WHITENING

CAVEX BITE & WHITE ABC MASTERKIT

-
Professional three step whitening system

AT-HOME

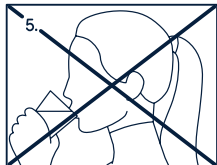
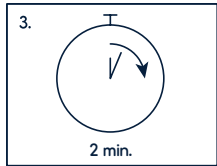
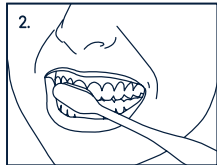
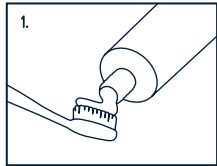
INSTRUCTIONS

CAVEX






CAVEX BITE&WHITE ABC MASTERKIT

Instructions for use

A Activate with Cavex Bite&White StainLess

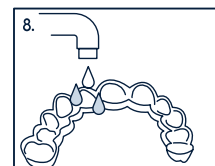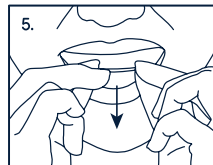
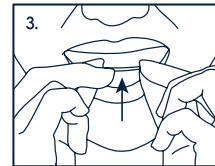
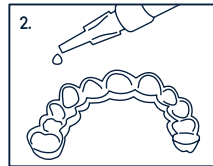
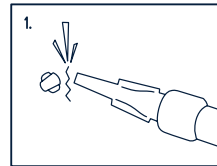
B Brighten with Cavex Bite&White Whitening Gel


Instruction video

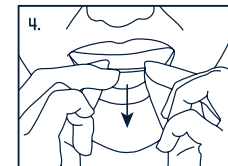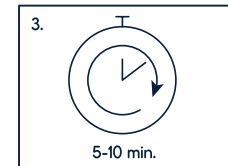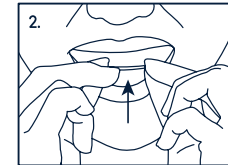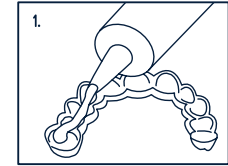C Condition with Cavex Bite&White ExSense

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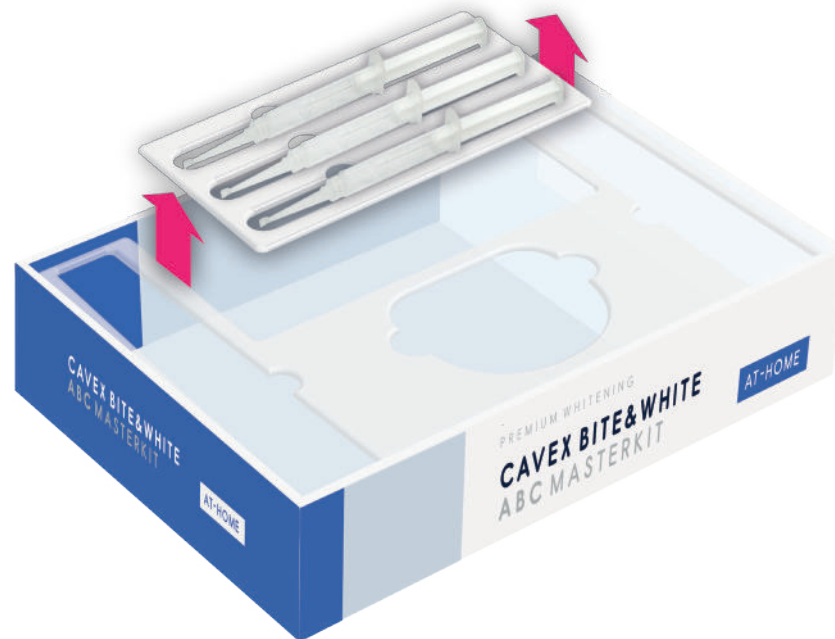Instructions Quick Reference Chart

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			C - CONDITION - CAVEX BITE&WHITE EXSENSE										
4	Keep the tray in the mouth for between 30 and 60 minutes	5/6/7	Rinse mouth thoroughly with water	8	Rinse whitening tray with cold water	1	Dispense Cavex Bite&White ExSense to the tray	2	Insert the whitening tray into the mouth	3	Keep the tray in the mouth for 5 to 10 minutes	4/5	After 5 to 10 minutes, spit out any excess material. Do not rinse with water!

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Cavex Bite&White ABC Master Kit :
3-step Home Whitening System

**STEP A = ACTIVATION:
CAVEX BITE&WHITE STAINLESS**

DESCRIPTION

Teeth Whitening Activator & Stain Removal Paste. Removes surface stains and plaque from teeth. Enhances whitening treatments by optimizing the pH in the mouth.

- Mildly abrasive to enamel / dentin
- High cleaning and polishing values.
- Increases mouth pH as pre-treatment for teeth whitening.
- Not intended for regular use.

PACKAGING
Tube 35 ml – 50 g

CAVEX BITE&WHITE STAINLESS INGREDIENTS

Aqua, Sorbitol, Hydrated silica, Glycerin, Erythritol, Lithium magnesium sodium silicate, Poloxamer, Sodium hexametaphosphate, Cellulose gum, Xanthan gum, Phenoxyethanol, Sodium saccharin, Mint aroma, Limonene, CI 42090

STEP B = BRIGHTEN: CAVEX BITE&WHITE

DESCRIPTION

Hydrogen peroxide is the bleaching agent in Cavex Bite&White. It is added as carbamide peroxide in order to obtain a sufficiently stable product. The carbamide peroxide breaks down rapidly into oxygen radicals and water. During the whitening process the free radicals attack the double carbon bonds that are the source of most colours (stains) in the tooth.

PACKAGING

- Cavex Bite&White 3 ml syringes

CAVEX BITE&WHITE ACTIVE INGREDIENTS

- 16 % Carbamide Peroxide – equivalent to 6 % hydrogen peroxide
- 0.2 % Potassium nitrate desensitizer
- 0.1 % Sodium fluoride desensitizer

COMPOSITION

Carbamide peroxide, Potassium nitrate, Sodium fluoride, Carbomer, Glycerin, EDTA, Potassium hydroxide, Peppermint oil, Water.

INDICATIONS FOR USE

Cavex Bite&White is a tooth whitening gel for use in lightening the colour of natural teeth. Cavex Bite&White should only be dispensed by or on the order of a dental professional. Before any whitening procedure, a consultation with a qualified dentist is

recommended to ensure no underlying oral or health issues.

CONTRAINDICATIONS

If it is recommended to consult a doctor, in case of:

- Pregnant and lactating women
- Those currently treated for a serious illness
- Children under the age of 18 years

CLINICAL PROCEDURE

- Record starting shade, review current dental health and make sure patient has realistic expectations about final results.
- Provisional or temporary restorations may discolour during the whitening process while composite or polycarbonate materials will remain unaffected. Therefore, patients with composite restorations must be advised that existing composites may require replacement, since only natural teeth will whiten.
- For patients requiring aesthetic or adhesive restorations, wait 2 weeks after the last whitening treatment to allow for colour stabilization. All restorations should be intact or well-sealed with adequate temporary materials.
- The use of a sensitivity relief toothpaste starting 2 weeks prior to the treatment procedure until 2 weeks after the treatment should be advised.
- Patients should always be warned of tooth sensitivity or gingival irritation during the treatment. The gingival areas that come in contact could turn red or white but will return to its normal colour after several days.
- To avoid soft tissue irritation, advise the patient to avoid overfilling trays, to wipe away any excess gel from the gum line and to rinse the mouth thoroughly after the treatment.
- After each treatment, make sure patient replaces the syringe cap.
- The patient should not eat, drink or smoke tobacco for at least 30 minutes after each treatment.
- Instruct the patient to stop the treatment and contact you immediately if any problem develops.
- The final shade will be achieved about 2 weeks after the start of the treatment. A follow-up appointment should be arranged after the treatment.

WARNINGS

- Contains Fluoride.
- Contains Hydrogen Peroxide.
- Avoid contact with eyes.
- Keep out of reach of children.
- Avoid exposure of known hypersensitive teeth.
- Do not eat, drink or smoke while wearing the filled custom-fitted trays.

- Harmful when swallowed.

FIRST AID INSTRUCTIONS

- If a large quantity (over 25% of the gel in the syringe) is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to: Sensitivity

Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment. Gingival Irritation Trays that extend onto the gingiva may cause mild to severe soft tissue irritation. If the trays fit properly, check if the patient applies the proper quantity in the proper way. The patient may need to reduce the amount of gel in their trays.

STORAGE ADVICE

- Store refrigerated (-8°C) in a dark, dry place.
- Do not use the material after the expiry date.

DISPOSAL

- Dispose of device in accordance with local waste regulations.

**STEP C = CONDITION:
CAVEX BITE&WHITE EXSENSE**

DESCRIPTION

Cavex Bite&White ExSense is a water-based gel with mint aroma that contains hydroxyapatite and potassium nitrate. Cavex Bite&White ExSense controls sensitivity and enhances whiteness of the teeth by hydroxyapatite remineralisation.

PACKAGING

Tube 42 ml – 50 g (sufficient for approximately 20 treatments)

CAVEX BITE&WHITE EXSENSE KEY INGREDIENTS:
Hydroxyapatite 2%, Potassium nitrate 0.1%

COMPOSITION

Aqua, Hydrogenated starch hydrolysate (Sorbitol), Glycerin, Erythritol, Lithium magnesium sodium silicate, Hydroxyapatite, Potassium chloride, Potassium nitrate, Phenoxyethanol, Xanthan gum, Tetrasodium pyrophosphate, Tartaric acid, Mint Aroma, Sodium saccharin.

INDICATIONS FOR USE

- After in-office and/or at-home tooth whitening
- After professional tooth cleaning
- Prevention and control of sensitivity

CONTRA-INDICATIONS

Do not use Cavex Bite&White ExSense in cases of allergies to mint aroma or any of the ingredients.

HANDLING PROCEDURE

1. Prepare and clean the teeth using tooth brush and floss.
2. Application:

Without Tray:

Wash hands thoroughly with soap and water. Dispense a small amount of Cavex Bite&White ExSense onto a clean finger. Gently apply gel on all surfaces of the upper and lower teeth. Allow gel to remain on tooth for 10 minutes. Spit out any excess gel and do not rinse mouth with water.

With Tray:

Dispense a small ribbon of Cavex Bite&White ExSense across the entire tray. Insert tray and gently rotate tray left and right to assure gel is distributed across teeth. Gently bite down on the tray and leave gel in place for 10 minutes. Remove tray, spit out any excess gel and do not rinse mouth with water. Rinse tray with cold water.

3. Always close the tube with the cap immediately after use.
4. For best results, do not drink or eat for at least 30 minutes after the application.
5. Use product every day for 10 days or as instructed by your dental professional.

WARNINGS & FIRST AID INSTRUCTIONS

In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention. Do not use the material after the expiry date.

STORAGE ADVICE

Store in a cool place, dry place (frost-free) and protected from direct sunlight. Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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