

# Instructions For Use

## Cavex Bite&White StainLess

### Product description

Before starting a dental whitening treatment, it is very important to properly clean the teeth to ensure an optimum whitening result.

Cavex Bite&White StainLess is a Teeth Whitening Activator & Stain Removal Paste. It removes surface stains and plaque from teeth. Besides it enhances whitening treatments by optimizing the pH in the mouth.

- Mildly abrasive to enamel / dentin
- High cleaning and polishing values.
- Increases mouth pH as pre-treatment for teeth whitening.
- Not intended for regular use.

### Composition

Aqua, Sorbitol, Hydrated silica, Glycerin, Erythritol, Lithium magnesium sodium silicate, Poloxamer, Sodium hexametaphosphate, Cellulose gum, Xanthan gum, Phenoxyethanol, Sodium saccharin, Mint aroma, Limonene, CI 42090

### Indication for use

The paste is used as a pre-treatment activator for in-office and/or at-home tooth whitening.

### Contra-indications & Warnings

#### Contra-indications

Do not use in cases of allergies to mint aroma or any of the ingredients.

#### Potential side effects

Hypersensitivities to the product or its components cannot be excluded in individual cases.

#### Storage Conditions

Do not use the material after the expiry date. Do not store at temperatures higher than 25 °C (77 °F). Avoid exposure to direct sunlight. Store in a cool and dry place(frost free). Always close the tube tightly after use. Keep out of reach of children.


#### Warnings

In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention. Do not use the material after the expiry date.


# Instructions For Use

## **Handling**


1. Dispense Cavex Bite&White StainLess onto toothbrush

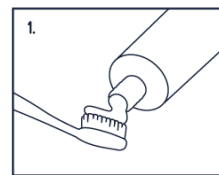

2. Brush the teeth thoroughly

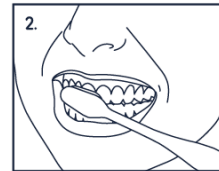


3. Clean teeth at least 2-3 minutes

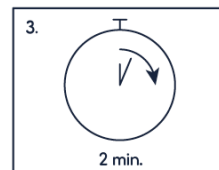

4. Spit out any excess material.



5. Do not rinse with water

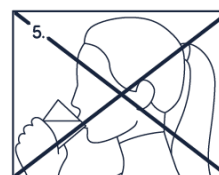

## **Labelling:**

This information is in compliance with Regulation (EC) 1223/2009 Concerning Cosmetic Products and also with European legislation.

## **Intended Patient(s)**

Cavex B&W StainLess can be used by anyone above the age of 18 as a pre-treatment for whitening gel.

For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

## **Intended User(s)**

Cavex B&W StainLess is mainly used by a dental professional. After consultation with a dental professional the product can also be used at home by the patient.

## **Clinical benefits**

The following clinical benefits can be considered for Cavex B&W StainLess:

StainLess paste cleans the teeth thoroughly, allowing the whitening gel to come into direct contact with the tooth enamel. It removes surface stains from the teeth and temporarily raises the pH value in the mouth. By doing so StainLess extreme cleaning paste optimizes hydrogen peroxide's performance.

# Instructions For Use

## **Claims**

- Thorough cleaning and removal of plaque
- Raises oral pH and optimized whitening conditions
- Fresh and pleasant taste

## **Product characteristics and specifications**

Every batch Cavex Bite&White StainLess is in compliance with our internal testing:

Test	Test method	Cavex Stainless Release Specification	
Appearance	B10	Blue paste	
Flavour	B11	Lime / mint flavor	
pH	B12	6.8 – 8.2	
Viscosity	B13	10 – 10000	Pa.s

## **Combination products:**

### **Accessories:**

Cavex B&W StainLess does not require specific combinations with accessories.

### **Combination products:**

Cavex B&W StainLess can be used in combination with a whitening treatment.

## **Storage advice**

Do not use the material after the expiry date. Do not store at temperatures higher than 25 °C (77 °F). Avoid exposure to direct sunlight. Store in a cool and dry place (frost free). Always close the tube tightly after use. Keep out of reach of children.

## **Disposal information**

Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system.

Cavex Holland BV  
Fustweg 5  
2031CJ Haarlem  
The Netherlands  
T +31 23 5307700  
dental@cavex.nl

Made in The Netherlands

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

Version: ZB713.00

Date: 12-9-2023