

**INSTRUCTIONS FOR USE****Indications**

Cavex Outline, Impression Paste Eugenol Free is a material for making impressions of the edentulous jaw. The mixed impression paste flows easily under light pressure, preventing distortions of the soft tissue occurring while the impression is made and accurately reproducing all details. For this reason, Cavex Outline, Impression Paste Eugenol Free is particularly suitable for rebasing and for the second impression with the individual tray. It does not contain eugenol and is therefore highly patient-friendly.

**Dosage**

Cavex Outline, Impression Paste Eugenol Free is supplied as a two-paste system in the contrasting colours blue and white. Squeezing out lines of equal length will dose the pastes in the correct proportion for mixing: 1 g of blue paste to 2,2 g of white paste.

Note: in all other proportions, setting will take longer.

**Mixing time**

Mix the two pastes well on the mixing block supplied using a sturdy stainless steel spatula until the mixture acquires a homogenous colour. This takes about 30 seconds.

The spatula can best be cleaned with a tissue before setting.

**Taking an impression**

Apply the prepared paste to the prosthesis or the impression tray and place this in the mouth under slight pressure within 1½ minutes of mixing the paste. The setting time in the mouth is 2-3 minutes. Remove the tray from the mouth and carefully rinse the impression with cold running water.

**Casting with plaster**

The impression can immediately be cast with plaster. The impressions remain dimensional stable up to 7 days.

**Properties**

The typical values for Cavex Outline Impression Paste Eugenol Free, measured according to ADA Specification No.16, are:

Consistency	33	mm
Initial setting time	3	min.
Final setting time	4	min.
Penetration hardness	1,3	mm
Cast surface		complies

**Storage:**

Store below 25°C

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

■ Cavex Holland BV  
Fustweg 5 - 2031 CJ Haarlem - The Netherlands  
Tel +31 23 530 7700  
[dental@cavex.nl](mailto:dental@cavex.nl) - [www.cavex.nl](http://www.cavex.nl)

Version: ZA722.03

Date: 10 December 2020