

# Cavex Alginate Mixer 3

## Instructions

***Always premix manually before mixing!***

The Cavex Alginate Mixer 3 will decrease the setting time by 45 to 60 seconds.

Program	M1	M2	M3	M4
Material properties	Extra Fast Set	Fast Set	Normal Set	Stone/Gypsum
Material volume	1 - 4 scoops	3 - 4 scoops	3 - 4 scoops	75 grams*
Maximum rotation speed	3.200 rpm	3.200 rpm	3.200 rpm	2.200 rpm
Program stages	Fast start - 6 sec. mixing - Fast stop - End	Fast start - 8 sec. mixing - Slow stop - End	Slow start Premix 8 sec. mixing - Slow stop - End	Fast start - 8 sec. mixing - Fast stop - End

\* Dose according to manufacturer's instructions and premix for 10 sec. so all powder is in contact with water.

