

-
PREMIUM WHITENING

CAVEX BITE&WHITE

3 SYRINGES

-
Whitening System

AT-HOME

INSTRUCTIONS

CAVEX

CAVEX BITE&WHITE PROFESSIONAL WHITENING SYSTEM

Indications for use

Cavex Bite&White is a tooth whitening gel for lightening the colour of natural teeth.

Contraindications

It is recommended to consult a doctor for:

- Pregnant and lactating women.
- Those currently treated for a serious illness.
- Children under the age of 18 years.

First aid instructions

- If gel is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least fifteen minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

Whitening procedure

- Review current dental health and make realistic expectations about final results.
- Provisional or temporary restorations may discolour during the whitening process while composite or polycarbonate materials will remain unaffected. Composite restorations may require replacement, since only natural teeth will whiten.
- For aesthetic or adhesive restorations, wait 2 weeks after the last whitening procedure to allow for colour stabilization. All restorations should be intact or well sealed with adequate temporary materials
- It is recommended to use a sensitivity relief product 2 weeks prior to, and 2 weeks after the whitening treatment.

Potential adverse effects

- Trays that extend onto the gingiva may

cause mild to severe soft tissue irritation. If the tray fits properly, check the proper gel quantity.

- Increased tooth sensitivity to cold or non-specific sensitivity in the teeth, gums, tongue, lips or throat can occur during the treatment. In general, symptoms should disappear within a few days after the treatment. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue.

Disposal

Dispose of device in accordance with local waste regulations.



see separate sticker

 **Cavex Holland BV**
Fustweg 5
2031 CJ Haarlem
The Netherlands
+31 23 530 77 00
dental@cavex.nl
cavex.nl