

PREMIUM IMPRESSION MATERIALS

CAVEX SILICONA BITE

-

Perfect the first time,
every time!



YOUR
IMPRESSION
IS OUR
SPECIALTY

CAVEX

PREMIUM IMPRESSION MATERIALS

CAVEX SILICONA BITE

A good bite registration material should be comfortable and easy to use. Ideally, it should deliver a perfect bite registration on the very first attempt. This is only achievable if the material is user-friendly, easy to handle, and 100% accurate.

Cavex SiliconA Bite more than meets all the key criteria for an ideal bite registration material. It offers a very short working and setting time of just 30 seconds each. In addition, it is highly filled, achieving a hardness of Shore D 40 (comparable to Shore A >94). Thanks to this ideal combination of properties and its excellent thixotropy, Cavex SiliconA Bite ensures a perfect bite registration.

The high hardness (Shore D 40) guarantees an extremely accurate registration of the occlusion, which retains its precision over time. Cavex SiliconA Bite is easy to trim and grind, making it especially convenient for dental technicians to work with.

With a short working and setting time, the total chair time for a bite registration procedure is only one minute. This significantly increases patient comfort and minimizes the risk of inaccuracies due to rapid stabilization.

An essential step in the procedure is the application of the material to the teeth. Thanks to special bite registration tips, a uniform and homogeneous layer of Cavex SiliconA Bite is applied to the bite surface. The thixotropic properties ensure the material stays exactly where it is placed.

Cavex SiliconA Bite flows easily from the cartridge but remains stable immediately after application. This ensures the bite surface is fully covered and the material is distributed evenly.

Cavex SiliconA Bite contains fillers with high reflectivity, making the material highly scannable for CAD/CAM applications.


Assortment:

AS54 Cavex SiliconA bite 2 x 50 ml cartridges

For more information, please contact Cavex:
Phone: +31 23 530 77 00
Website: www.cavex.nl

Features

- Superior hardness
- Fast
- Highly thixotropic
- Scannable
- Unique dispensing tips

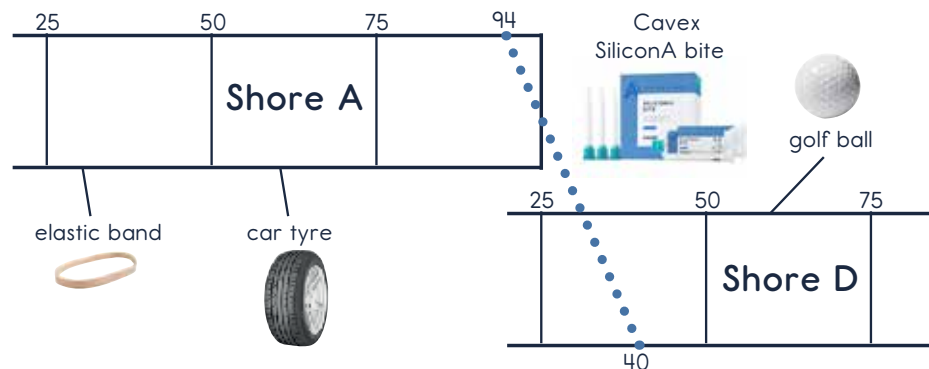

YOUR IMPRESSION IS OUR SPECIALTY

CAVEX

Fustweg 5 | 2031 CJ Haarlem | The Netherlands | cavex.nl

ZYA75E.01