

Cavex Boil&Bite

ONE BITE FITS ALL

Revolutionary

This *thermoplastic tray* is an **upper and lower tray in one** and has a number of different uses.

This revolutionary tray is well tolerated in the mouth and is suitable for everybody!

Exceptionally easy

Either the dentist or the patient can make the Cavex Boil&Bite themselves.

1. Bring some water to the boil.
2. Place the Cavex Boil&Bite in a suitable (heat-resistant) glass.
3. Add enough water to the Cavex Boil&Bite to cover the tray – with the exception of the handle. Leave the unit submerged in the water for one minute.
4. Carefully remove the Cavex Boil&Bite from the hot water.
5. Shake off any surplus water.
6. Put the Cavex Boil&Bite into your mouth immediately and adjust the shape of the tray by pressing it firmly against the teeth with your fingers. Do this for both the top and bottom teeth.



Multi-purpose

This custom-made tray is ideally suited to various purposes:

- **Whitening treatments**
- **Fluoride treatments**
- **Tooth sensitivity treatments**

The imprint of the teeth in the tray means that the various gels remain well positioned. The teeth can be whitened with Cavex Bite&White and then effectively **conditioned and desensitised** with Cavex Bite&White ExSense.

8 pieces in 1 box



■ **Cavex Holland BV**
Fustweg 5
2031 CJ Haarlem
The Netherlands

T +31 23 530 77 00
dental@cavex.nl
www.cavex.nl