Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
Nederland
+31 (0)23 530 77 00
dental@cavex.nl
www.cavex.nl

INSTRUCTIONS

CONTENTS

English	1
Español	9
Deutsch	17
Français	25

CAVEX

Bite&White

Impact

Professional Home Whitening System

Step by step instruction inside cover

STEP 1.


www.cavex.nl

www.cavex.nl

STEP 2.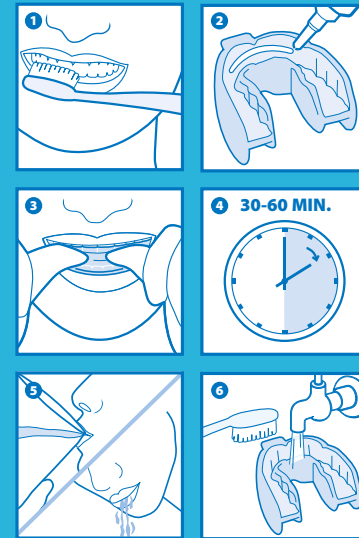

Congratulations with your purchase of Cavex **Bite&White** Impact.

Before beginning this exciting cosmetic journey, we would like to ask you to **please read these instructions attentively**. This product consists of 2 separate steps. First you need to make your own whitening tray with the Cavex Boil&Bite. Second the use of the tray with Cavex Bite&White Impact.

STEP 1. (Left-hand section inside cover)

Make your own whitening tray with Cavex Boil&Bite

1. Bring some water to the boil.
2. Place the Cavex Boil&Bite in a suitable (heat-resistant) glass.
3. Add enough water to the Cavex Boil&Bite to cover the tray – with the exception of the handle. Leave the tray submerged in the water for one minute.
4. Carefully remove the Cavex Boil&Bite from the hot water.
5. Shake off any surplus water.
6. Put the Cavex Boil&Bite into your mouth immediately and adjust the shape of the tray by pressing it firmly against the teeth with your fingers. Do this for both the top and bottom teeth.


STEP 2. (Right-hand section inside cover)

GB

Whiten your teeth with the Cavex Boil&Bite tray and Cavex Bite&White Impact

1. Brush your teeth thoroughly.
2. Apply a thin line of Cavex Bite&White Impact whitening gel **on both sides** at the front of the tray.
3. Insert the Cavex Boil&Bite tray into the mouth.
4. Keep the tray in the mouth for at least 30 minutes and a maximum of 60 minutes.
5. Rinse mouth with water thoroughly.
6. Clean the tray using a toothbrush.

Cavex Boil&Bite

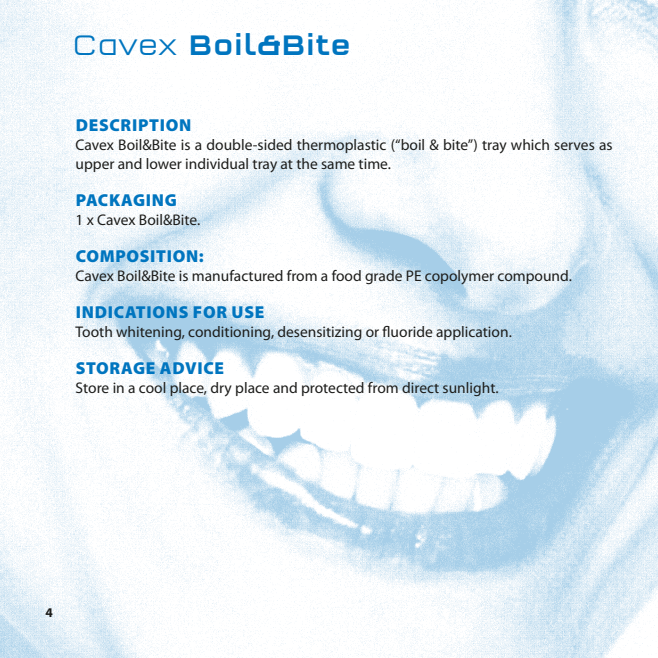DESCRIPTION

Cavex Boil&Bite is a double-sided thermoplastic ("boil & bite") tray which serves as upper and lower individual tray at the same time.

PACKAGING

1 x Cavex Boil&Bite.

COMPOSITION:

Cavex Boil&Bite is manufactured from a food grade PE copolymer compound.

INDICATIONS FOR USE

Tooth whitening, conditioning, desensitizing or fluoride application.

STORAGE ADVICE

Store in a cool place, dry place and protected from direct sunlight.

DESCRIPTION

Hydrogen peroxide is the bleaching agent in Cavex Bite&White Impact. It is added as PVP-peroxide in order to obtain a sufficiently stable product. The PVP-peroxide breaks down rapidly into oxygen radicals and water. During the whitening process the free radicals attack the double carbon bonds that are the source of most colours (stains) in the tooth.

PACKAGING

Cavex Bite&White Impact 40 gram (30ml) tubes.

COMPOSITION

Propylene glycol, PVP-peroxide, Polyglycerin, Peppermint oil, Potassium nitrate.

Active ingredients: PVP-peroxide (5- 6 % hydrogen peroxide) - Potassium nitrate desensitizer (<0,1%)

INDICATIONS FOR USE

Cavex Bite&White Impact is a tooth whitening gel for use in lightening the colour of natural teeth. Cavex Bite&White Impact should only be dispensed by or on the order of a dental professional. Before any whitening procedure, a consultation with a qualified dentist is recommended to ensure no underlying oral or health issues.

CONTRAINDICATIONS

It is recommended to consult a doctor, in case of:

- Pregnant and lactating women
- Those currently treated for a serious illness
- Tooth whitening is not allowed under the age of 18

CLINICAL PROCEDURE

- Record starting shade, review current dental health and make sure patient has realistic expectations about final results.
- Provisional or temporary restorations may discolour during the whitening process while composite or polycarbonate materials will remain unaffected. Therefore, patients with composite restorations must be advised that existing composites may require replacement, since only natural teeth will whiten.
- For patients requiring aesthetic or adhesive restorations, wait 2 weeks after the last whitening treatment to allow for colour stabilization. All restorations should be intact or well sealed with adequate temporary materials.
- The use of a sensitivity relief toothpaste starting 2 weeks prior to the treatment procedure until 2 weeks after the treatment should be advised.
- Patients should always be warned of tooth sensitivity or gingival irritation during the treatment. The gingival areas that come in contact could turn red or white but will return to its normal colour after several days.
- To avoid soft tissue irritation, advise the patient to avoid overfilling trays, to wipe away any excess gel from the gum line and to rinse the mouth thoroughly after the treatment.
- After each treatment, make sure patient replaces the syringe cap.
- The patient should not eat, drink or smoke tobacco for at least 30 minutes after each treatment.
- Instruct the patient to stop the treatment and contact you immediately if any problem develops.
- The final shade will be achieved about 2 weeks after the start of the treatment. A follow-up appointment should be arranged after the treatment.

WARNINGS

- Contains Hydrogen Peroxide.
- Avoid contact with eyes.
- Keep out of reach of children.
- Avoid exposure of known hypersensitive teeth.
- Avoid smoking and strong colorants within 24 hours after the whitening treatment.
- Harmful when swallowed.

FIRST AID INSTRUCTIONS

- If a large quantity is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White Impact include, but are not limited to:

Sensitivity: Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat.

In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

Gingivale Irritation: Trays that extend onto the gingiva may cause mild to severe soft tissue irritation. If the trays fit properly, check if the patient applies the proper quantity in the proper way. The patient may need to reduce the amount of gel in their trays.

STORAGE ADVICE

- Room temperature (<25°C).
- Do not use the material after the expiry date.
- Shelf life 3 years.

DISPOSAL

- The packaging must be disposed of in accordance with local rules for waste disposal.

Made in The Netherlands.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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