GB - Instructions for use

Cavex ScanSpray

Before spraying, remove excess water. Shake the can very well before use.

Recommended distance to the impression approximately 10-15 cm (4-6 inches).

Apply the spray in a thin, even layer on the surface of the impression to be scanned.

Remove powder excess with compressed air.

After use clean with water and wipe the outlet opening of the spray cannula.

For professional use only.

Extra-oral application.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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