

Cavex Water Dosing Bottle

Several external factors may influence the quality of the alginate impression. It is therefore essential to standardise as many of these factors as possible. The Cavex Water Dosing Bottle is an accessory that is part of the solution to standardise external factors.

How to use

Squeeze the bottle to fill up the cap until the right amount of water.



Measurement in scoops

For the right amount of water per scoop of alginate, the lines on the cap can be used. For example with two scoops of alginate squeeze the bottle till the second line.



Adjustable straw

The straw in the bottle can be adjusted to the desired measurement. Then you will be sure of the same amount of water every time.



Conditioned water

With the water bottle you have a constant water temperature because it is always on room temperature. A constant water temperature = constant mixing and setting time = constant and predictable quality.

CAVEX YOUR IMPRESSION IS OUR CONCERN



Cavex Holland BV,
Fustweg 5
2031 CJ Haarlem
The Netherlands
Tel +31 23 530 77 00
dental@cavex.nl
www.cavex.nl

CAVEX
www.cavex.nl

ATI021.00