

# Whitening treatment

CAVEX



ONLY 15 MINUTES



CAVEX


## Bite & White

PROFESSIONAL HOME WHITENING SYSTEM


READY 2 USE

# User instruction


- 1 Brush the teeth thoroughly

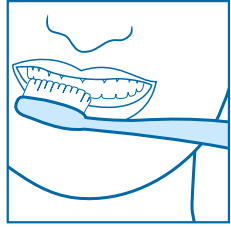


- 2 Open the foil as indicated on the packaging

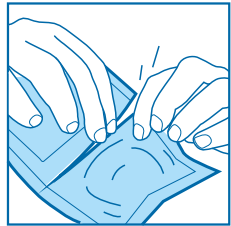


- 3 Apply the bleaching tray in the mouth

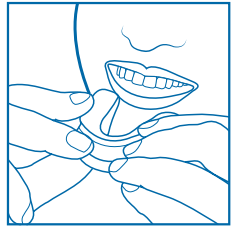

- 4 Wear the bleaching tray for fifteen minutes

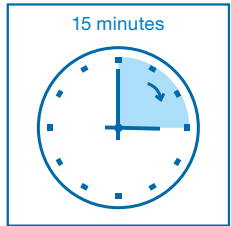

- 5 Rinse the mouth after bleaching

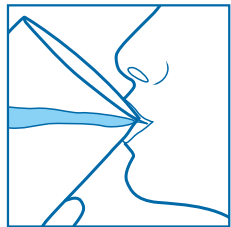

# HPS Technology

More and more people desire white teeth easily and quickly. Cavex offers the solution: **Bite&White Ready 2 Use**.

With one prefilled bleaching tray, the upper and lower teeth are bleached simultaneously, resulting in homogenous white teeth. This result is achieved with a treatment of only fifteen minutes for eight days!



*Before treatment*



*After treatment*

The Ready 2 Use bleaching trays are filled with a HPS (Hydrogen Peroxide Superior) gel developed and patented by Cavex. The HPS technology is a composition of very pure hydrogen peroxide of 6% in a gel with a neutral pH value.

This HPS technology offers unprecedented benefits:

- The product can be stored at room temperature. The gel is activated by contact with moisture.
- The bleaching tray needs to be worn in the mouth for just fifteen minutes.
- A full whitening treatment in just eight days with significantly whiter teeth as a result.



# CAVEX Bite&White

PROFESSIONAL HOME WHITENING SYSTEM

## READY 2 USE

Cavex Bite&White Ready 2 Use is available through your distributor in the following packagings:



**Single patient packaging**  
8 prefilled trays  
Art. nr. BW031



**Bulk packaging**  
50 prefilled trays  
Art. nr. BW035

For more information check: [www.cavex.nl/ready2use](http://www.cavex.nl/ready2use)

**CAVEX**  
[www.cavex.nl](http://www.cavex.nl)

**Cavex Holland BV**  
Fustweg 5  
2031 CJ Haarlem  
The Netherlands  
T +31 23 530 77 00  
[dental@cavex.nl](mailto:dental@cavex.nl)  
[www.cavex.nl](http://www.cavex.nl)

ZYH008E.04