

Recommended by professionals

Cavex Bite&White is a product by Cavex Holland, a specialised manufacturer of dental products with over 100 years of experience. Cavex Bite&White Professional Dental Whitening System is extensively tested and recommended by dentists and dental hygienists. Cavex Bite&White contains 16% carbamide peroxide, a material that is proven as a bleaching agent. It is effective without harming your teeth.

Fluoride

Cavex Bite&White contains sodium fluoride for strengthening tooth enamel. It also contains potassium nitrate which helps to relieve tooth sensitivity. Mint is added for a pleasant taste in the event some of the material escapes from the mould. To prevent dental or health problems and in order to use Cavex Bite&White, consultation with a qualified dentist or dental hygienist is necessary prior to bleaching. All Cavex Bite&White whitening materials are pH-neutral (pH = 6,2) and therefore safe for your teeth.

In addition to the patient kit, a refill is available which is sufficient for approximately ten bleachings.



More information

Your dentist can tell you more about Cavex Bite&White.



CAVEX BITE&WHITE

Sensitive teeth?

NEW! Cavex Bite&White ExSense T o o t h c o n d i t i o n e r

Cavex Bite&White ExSense is a tooth conditioner for use at home that **eliminates tooth sensitivity** and helps to prevent new sensitivity. It also makes the tooth surface smooth and shiny so that teeth look whiter.

Cavex Bite&White ExSense is also ideal for use after teeth have been whitened. The conditioner eliminates any sensitivity **and stimulates the quick recovery of your teeth.**

Ask your dentist if Cavex Bite&White ExSense is right for you.



CAVEX
Bite&White
Professional Dental Whitening System

Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
www.cavex.nl

WOULD YOU LIKE A BRIGHT WHITE SMILE?



Teeth
whitening
at home
under dental
supervision

pH-neutral,
so safe for
your teeth!

CAVEX
Bite&White
Professional Dental Whitening System

Bleach your teeth at home with Cavex Bite&White. Safe, effective and easy!

Bright white teeth

Beautiful white teeth look attractive. Something to be jealous of! Moreover, white teeth give you confidence. Did you know that it is now very easy to bleach your teeth? Cavex Bite&White is a new and safe product for bleaching your teeth at home. Cavex Bite&White works fast, is easy to use and leaves you smiling brightly again!

Teeth whitening safely and effectively

Most people start out in life with immaculate white teeth. Coffee, tea and other foods and drinks discolour teeth over time. In addition, the use of medicines or hereditary teeth problems can play a role.

Growing older also influences the colour of your teeth.

The protective enamel coating on your teeth becomes thinner, enabling colouring agents to penetrate even faster whereby the underlying, relatively darker dentine becomes more visible.

An obvious solution is teeth whitening. Cavex Bite&White has been developed for this purpose. A safe and effective teeth whitening agent, the treatment does not take place in a clinic or in the dental chair. You carry this out yourself, wherever and whenever you choose.*

* The result depends, among other things, on the nature of the discolouration, the intensity, the overall oral hygiene and the duration of the treatment.



Consult your dentist

Cavex Bite&White is only available through a dental professional. It is advisable to contact your dentist prior to bleaching. Your dentist can advise you on the possibilities for bleaching your teeth, offer useful advice and carry out important preparatory work such as a check-up, polishing your teeth and removing tartar. Your dentist can establish the correct colour of your teeth and, according to your requirements, determine a new colour. This colour will be the basis for the bleaching schedule.



Before treatment.



After treatment,
a bright white smile.

Bleaching teeth safely with a custom mould

A professionally made impression of your teeth is used to make a mould that fits your teeth perfectly. The mouthpiece moulds perfectly to the shape of your teeth and minimises the bleaching gel from coming into contact with your gums. The bleaching gel can cause some local irritation or sensitivity. Your dentist will indicate exactly how much gel you should use.



You will be given the transparent mouthpiece(s) and bleaching gel to take home with you.

Continue bleaching at home

By consulting a specialist first and allowing him to do the preparatory work you will be assured of minimum risks and maximum results. You can continue bleaching at home. You can compare the final result with the colour strip supplied by your dentist which shows the initial colour of your teeth.



How often should you bleach your teeth?

The bleaching routine is determined in consultation with your dentist. You will achieve the best results with Cavex Bite&White if you bleach for at least one hour a day during approximately 10 consecutive days. You can bleach during the day, whenever it suits you!



**Recommended
by dentists and
dental hygienists**

For optimal results, your teeth will be checked first to remove any tartar and then polished.