— PREMIUM DENTAL PRODUCTS

# CAVEX BITE & WHITE SHADE CORRECTION

IN-OFFICE

INSTRUCTIONS

Cavex Holland BV  
Fustweg 5  
2031 CJ Haarlem  
The Netherlands  
+31 23 530 77 00  
dental@cavex.nl  
cavex.nl

CE

CAVEX

CAVEX

— PREMIUM DENTAL PRODUCTS

# CAVEX BITE&WHITE SHADE CORRECTION

IN-OFFICE

INSTRUCTIONS

— PREMIUM DENTAL PRODUCTS

## CAVEX BITE&WHITE SHADE CORRECTION

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# CAVEX BITE&WHITE SHADE CORRECTION

- Professional shade recovery product

## Indications for use

Cavex Bite&White Shade Correction is a single use in-office product for the shade correction of vital and non-vital teeth in just 30 minutes.

## Contraindications

It is recommended to consult a doctor for:

- Pregnant and lactating women.
- Those currently treated for a serious illness.
- Persons under the age of 18 years.

## Clinical procedure

- Pre-treatment examen the current dental health and make realistic expectations about final results.  
 - During the treatment composite or polycarbonate materials remain unaffected. Therefore composite restorations may require replacement after treatment, since only natural teeth will be affected.  
 - For aesthetic restorations, wait two weeks after the last treatment to allow color stabilization.  
 - All restorations should be intact or well sealed with adequate temporary materials.  
 - It is recommended to use a desensitizer (i.e. Cavex ExSense) two weeks prior to, and two weeks after the treatment.  
 - Determine the initial tooth shade and polish the teeth with a prophylaxis angle and prophylaxis paste to remove plaque.

## Barrier procedure

- Insert a cheek retractor and dry the gingiva with air.
- Apply the barrier to the gingival tissue along the margins of the tooth.
- Cure for 20 sec. with a curing light.

Cavex Bite&White Shade Correction gel is based on active oxygen which is added as stabilized, encapsulated hydrogen peroxide. On contact with saliva, the hydrogen peroxide breaks down rapidly into oxygen and water. During this process, free radicals arise which attack the double carbon bonds that are the source of most discoloring (stains) of the teeth.

## Correction procedure

- Apply a thin layer of Cavex Bite&White Shade Correction gel onto the teeth.
- After ten to fifteen minutes remove the gel by suction.
- Repeat the application and removal steps two to three times.
- Rinse the teeth with water and suction.

## Post-treatment procedure

- Determine the "After tooth shade".
- Advise the patient to use a sensitivity relief agent (i.e. Cavex ExSense) for at least two week after treatment.

## Potential adverse effects

- Gel that comes into contact with the gingiva may cause severe soft tissue irritation.
- Increased tooth sensitivity to cold or non-specific sensitivity in the teeth, gums, tongue, lips or throat may occur during the treatment. In general, symptoms should disappear within a few days after the treatment. The treatment can be proceeded with shorter, less frequent applications until the desired result has been achieved. If the discomfort remains, discontinue the treatment.

## Disposal

Dispose of the product in accordance with local waste regulations.

## Safety advice

- If gel is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least fifteen minutes and

contact a doctor.

- If gel gets into contact with clothing, skin or hair, remove contaminated clothing and flush skin or hair with running water.

## Storage advice

- Store in a cool, dry place (frost-free) and protected from direct sunlight.
- Do not use the material after the expiry date.

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