

# Instructions For Use

## Cavex GreenClean

### Product description

Cavex GreenClean contains a complex forming agent which is able to break down the organic matrix of the alginate impression material, thus making it easy to rinse or brush it off the tray or spatula. Cavex GreenClean is pH neutral and does not work aggressively to trays, spatulas and other instruments.

### Composition

Function	Raw Material	% Mass
Former	Sodium	99
Colour	Pigment Brilliant blue 1	< 0,5
Colour	Pigment Tartrazine	< 0,5
Flavour	Natural lemon fragrance	0,1
Total		100

### Indication for use

Use **Cavex GreenClean** exclusively for removing alginate or plaster residues from impression trays, spatulas, flasks or other instruments.

### Warnings

Cavex GreenClean is tested under internal product specifications:

- Regulation (EC) 1272/2008 (CLP regulation).
- Biodegradable (Zahn-Wellens Test).

Cavex GreenClean has no dangerous substances.

### Handeling

- Dissolve 2 measuring spoons (50 grams) of **Cavex GreenClean** in 0.5 litres of lukewarm water.
- Remove as much as possible of the alginate and plaster residue from the tray, spatula or flask.
- Immerse the tray, spatula or flask in the solution for 15 to 30 minutes.
- Remove the tray, spatula or flask from the solution and brush away any residue that still remains. Rinse with water.

### Labelling:

This information is in compliance with Regulation (EC) 1272/2008.

# Instructions For Use

## **Intended User(s)**

Cavex GreenClean will only be used by dental professionals (dentist, dental technician).

## **Claims**

- PH neutral
- Skin-friendly
- Biodegradable
- 1 kg> 500 clean impression trays
- Refreshing smell

## **Product characteristics and specifications**

Cavex GreenClean is tested under internal product specifications:

Characteristic	Internal specifications	Cavex GreenClean
Appearance	Green powder, lemon flavour	Green powder, lemon flavour
Colour	Clear green solution	Clear green solution

## **Storage advice**

Do not store at temperatures higher than 25°C (77°F). Do not use this material beyond its expiry date. Avoid direct exposure to sunlight. Close containers firmly after use and keep out of reach of children.

## **Disposal information**

Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system.

Cavex Holland BV  
Fustweg 5  
2031CJ Haarlem  
The Netherlands  
T +31 23 5307700  
dental@cavex.nl

Made in The Netherlands

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

Version: HA001IFU.00

Date: 9-5-2023