

# THE SECRET OF DETAILED JAW IMPRESSIONS

## Discover the 'Sandwich Layering' Technique of Alginate

Taking an accurate impression of an edentulous jaw is essential for creating comfortable and well-fitting dentures. The innovative 'sandwich layering' technique offers you the ability to make impressions that are just as detailed as those made with silicone. By using the liquid Cream alginate in combination with the stiffer CA37 alginate in the impression tray, you achieve a perfect balance between flexibility and stability. This method ensures that all anatomical details of the jaw are accurately captured, resulting in an optimal fit and a satisfied patient.

### STEP-BY-STEP GUIDE

#### Step 1: Preparation

Ensure that the surface is clean. Rinse with water, and dry any excess moisture.

#### Step 2: Mixing the Alginate

- Mix the Cavex CA37 Normal set alginate as described on the packaging and apply it in a tray.
- Immediately mix the Cavex Cream Fast set alginate, or if more time is needed, the Normal set as described on the packaging, and place it in a syringe.



#### Step 3: Applying the Alginate

Gently place the syringe with Cream alginate in the mouth and apply an even amount around the surface of the jaw. Ensure that the alginate makes good contact with the jaw.




#### Step 4: Applying the Impression Tray

Place the tray with CA37 alginate in the patient's mouth over the layer of applied Cream alginate. Allow the alginate to set for the prescribed time.

#### Step 5: Removing the Impression

After the alginate has set, carefully remove the impression from the mouth. Check if all desired details are clearly visible.

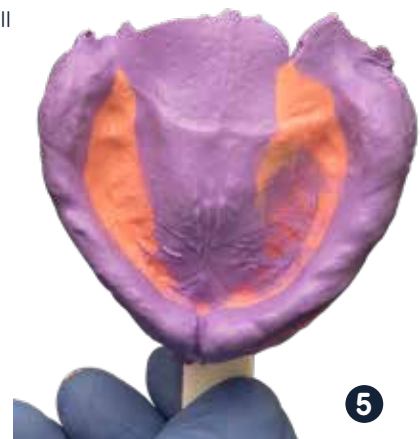

**YOUR  
IMPRESSION  
IS OUR  
SPECIALTY**