CAVEX EXSENSE

PRODUCT DESCRIPTION

Cavex ExSense is the perfect solution for sensitive teeth. Sensitive teeth often occur as a result of dental erosion or exposed dentin. Cavex ExSense is based on 2% hydroxyapatite that seals the tubules and microcracks in the enamel using a hydro-dispersing sol. In doing so, it completely closes off sensitive areas. The addition of 0.1% potassium nitrate also restores the micro-hardness of the elements. In addition to complete closure, crystallization takes place which supports and accelerates the remineralization process. Cavex ExSense is available in a tube (42 ml – 50 grams) and in a convenient brushpen (6 ml – 7 grams). The product can be applied in various ways, depending on the nature and extent of the sensitivity. The most effective method is to use a custom-made tray filled with Cavex ExSense in a tube, allowing the teeth to 'bathe' for 10 minutes. For more localized sensitivity, Cavex ExSense can be applied directly to the affected area with the tip of a finger or the with brushpen. The brushpen's fine applicator makes it especially easy to target specific spots with accuracy and control.

COMPOSITION

Water, sorbitol, glycerin, erythritol, lithium magnesium sodium silicate, hydroxyapatite, potassium nitrate, phenoxyethanol, xanthan gum, tetrasodium pyrophosphate, tartaric acid, spearmint oil, sodium saccharin.

HANDLING

Clean teeth beforehand by brushing and flossing.

Application with tray (tube):

Dispense a small amount of Cavex ExSense across the entire tray. Place the tray in the mouth and move the tray a little from left to right to spread the gel on the teeth. Gently bite down on the tray and leave the tray with the gel in place for 10 minutes. Remove the tray and spit out any excess gel but do not rinse afterwards. Rinse the tray with cold water.

Application with fingertip (tube):

Wash hands thoroughly with water and soap. Dispense a small amount of Cavex ExSense on a clean finger and apply it on the desired spot. Leave the gel on for 10 minutes. Spit out any excess gel but do not rinse afterwards.

Application with brushpen:

Twist the knob until the product emerges from the brush tip. Apply the product locally to the desired spot. Leave the gel on for 10 minutes. Spit out any excess gel but do not rinse afterwards.

- For best results: do not drink or eat for at least 30 minutes after application.
- Use Cavex ExSense once a day for a minimum of 10 days or follow the instructions of your dental professional.

INDICATION FOR USE

- Prevention and reduction of tooth sensitivity.
- After teeth whitening in dental practice and / or at home.

CONTRA-INDICATIONS & WARNINGS

Contra-indications:

Avoid using Cavex ExSense if you are allergic to mint or any of its other ingredients. For pregnant or breastfeeding women, it is important to carefully weigh the potential risks of the treatment against its benefits, keeping in mind the health of the unborn baby or infant. Consultation with a doctor is strongly recommended before use.

Potential side effects:

Hypersensitivities to the product or its components cannot be excluded in individual cases.

Warnings:

In case of contact with the eyes, rinse thoroughly with plenty of water and seek medical attention.
Do not use the product after expiry date

CLAIMS

- Instant and long-lasting relief from sensitive teeth
- Helps to prevent sensitive teeth
- Accelerates and reinforces the process of remineralization


CAVEX EXSENSE

STORAGE ADVICE

Store at temperatures between 0 °C (32 °F) to 25 °C (77 °F).

Store in a cool and dry place (frost-free) and protected from direct sunlight.

Always close the tube or brushpen tightly after use. Keep out of reach of children.

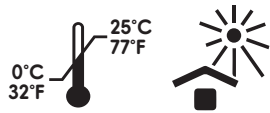DISPOSAL INFORMATION

Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

 Cavex Holland BV
Fustweg 5
2031CJ Haarlem
The Netherlands
T +31 23 5307700
dental@cavex.nl
cavex.nl

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