CAVEX FRESH4SURE

PRODUCT DESCRIPTION

Cavex Fresh4Sure is a mouthwash that helps support fresh breath and reduce unpleasant oral odours. Its carefully selected ingredients – including zinc salts and erythritol – contribute to a clean and fresh mouthfeel. The formula also contains a muco-adhesive polymer and a humectant, which help maintain moisture and comfort in the mouth. A refreshing blend of minty aromas provides a pleasant taste and a long-lasting fresh breath.

COMPOSITION

Aqua, polyglycerin-3, sorbitol, erythritol, poloxamer 407, sodium citrate, PEG40, mentha piperita oil, PVM/MA copolymer, sodium benzoate, zinc citrate, xanthan gum, sodium saccharin, acid blue 9, ethyl menthane carboxamide, menthol/eucalyptol.

INSTRUCTIONS

- Open the bottle and fill the cap with 10 ml liquid.
- Rinse and gargle for 30 seconds.
- Do not swallow but spit out the residue. Do not rinse with water.

WARNINGS

Do not swallow! For adults and children over 12 years old.

STORAGE ADVICE

Store at room temperature (<25°C). Shelf life unopened: 36 months. To be used within 12 months of opening. Do not use the product after the expiry of the recommended shelf life.

DISPOSAL INFORMATION

Dispose of device in accordance with local waste regulations.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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