—
PREMIUM DENTAL PRODUCTS

CAVEX BITE & WHITE SHADE CORRECTION

IN-OFFICE

INSTRUCTIONS

—
Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
The Netherlands
+31 23 530 77 00
dental@cavex.nl
cavex.nl

ZB718.04
CE

CAVEX

CAVEX

CAVEX BITE&WHITE SHADE CORRECTION

IN-OFFICE

INSTRUCTIONS

CAVEX BITE&WHITE SHADE CORRECTION

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CAVEX BITE&WHITE SHADE CORRECTION

Professional shade recovery product

Cavex Bite&White Shade Correction gel is based on active oxygen which is added as stabilized, encapsulated hydrogen peroxide. On contact with saliva, the hydrogen peroxide breaks down rapidly into oxygen and water. During this process, free radicals arise which attack the double carbon bonds that are the source of most discoloring (stains) of the teeth.

INDICATIONS FOR USE

Cavex Bite&White Shade Correction is a single use in-office product for the shade correction of vital and non-vital teeth in just 30 minutes.

CONTRAINDICATIONS

- It is recommended to consult a doctor for:
- Pregnant and lactating women.
- Those currently treated for a serious illness.
- Persons under the age of 18 years.

CLINICAL PROCEDURE

- Pre-treatment examen the current dental health and make realistic expectations about final results.
- During the treatment composite or polycarbonate materials remain unaffected. Therefore composite restorations may require replacement after treatment, since only natural teeth will be affected.
- For aesthetic restorations, wait two weeks after the last treatment to allow color stabilization.
- All restorations should be intact or well sealed with adequate temporary materials.
- It is recommended to use a desensitizer (i.e. Cavex ExSense) two weeks prior to, and two weeks after the treatment.
- Determine the initial tooth shade and polish the teeth with a prophylaxis and prophylaxis paste to remove plaque.

BARRIER PROCEDURE

- Insert a cheek retractor and dry the gingiva with air.
- Apply the barrier to the gingival tissue along the margins of the tooth.
- Cure for 20 sec. with a curing light.

CORRECTION PROCEDURE

- Moisten the teeth before application.
- Apply a generous layer of Cavex Bite&White Shade Correction (at room temperature) gel to the teeth. Use the dosing lines on the pen as a reference. One pen can be used for three cycles.
- After ten to fifteen minutes remove the gel by suction.
- Repeat the application and removal steps two to three times.
- After the final cycle, rinse the teeth thoroughly with water and suction.

Important: Always ensure the teeth are properly moistened after removing the gel and before applying a new layer.

POST-TREATMENT PROCEDURE

- Determine the "After tooth shade".
- Advise the patient to use a sensitivity relief agent (i.e. Cavex ExSense) for at least two week after treatment.

POTENTIAL ADVERSE EFFECTS

- Gel that comes into contact with the gingiva may cause severe soft tissue irritation.
- Increased tooth sensitivity to cold or non-specific sensitivity in the teeth, gums, tongue, lips or throat may occur during the treatment. In general, symptoms should disappear within a few days after the treatment. The treatment can be proceeded with shorter, less frequent applications until the desired result has been achieved. If the discomfort remains, discontinue the treatment.

DISPOSAL

Dispose of the product in accordance with local waste regulations.

SAFETY ADVICE

- If gel is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least fifteen minutes and contact a doctor.
- If gel gets into contact with clothing, skin or hair, remove contaminated clothing and flush skin or hair with running water.

STORAGE ADVICE

- Store refrigerated when not in use.
- Short transport (<1 month) is possible under standard conditions.
- Keep away from heat sources and direct sunlight.

Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty. This also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of our advice, any damage occur our liability is limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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